

# NarcoCheck

## ID-Test®

### Drug Purity Test

www.narcocheck.com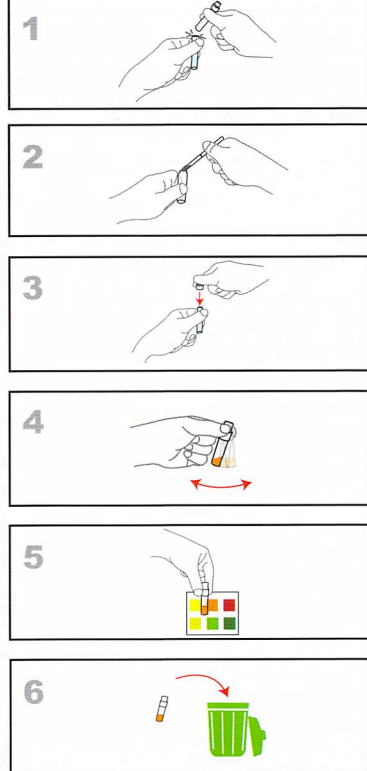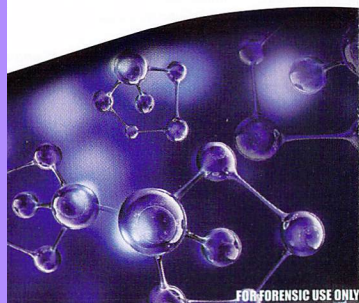

homogeneous.

9. Compare the color change (if any) with the colors on the color chart included in the leaflet.

For MDMA, this leaflet provides one color chart usable 5 minutes after the test is done, and one color chart usable 30 minutes after the test, because the colors change quickly. Compare the color charts to the top layers color of the test, as MDMA will not color the bottom layers, which can vary depending on the cutting agents contained in the drug.

10. You can then dispose of everything together into a rubbish bin without any harm to the environment.

#### Disclaimer:

Regardless of the result obtained, extreme vigilance is required, as:

- The sample may contain unidentified harmful substances.
- It is possible that a procedural or handling error, as well as the unexpected interference of certain substances, such as certain cutting products, may cause an erroneous result. The use of this test must be done with the full knowledge that no rapid test can be 100% reliable.
- The results obtained by this test must be confirmed by a reference analytical method such as GC/MS and HPLC-MS/MS.

For these reasons, NarcoCheck® cannot be held responsible for any damage or accident that may result from the use of this test.

#### Important note:


This test is a professional device to help evaluate presumptive samples of drugs or narcotics in the form of powders (fine or agglomerated), crystals or tablets. It is not intended to test «anything and everything» (food products, beverages, biological materials, etc.), and in particular everyday products (household, cosmetics, gardening, etc.), which contain chemical compounds that can cause unexpected color reactions.

#### The possession and use of drugs is prohibited by law.

#### Storage advice:

- Keep below <35°C (i.e. 95°F).
- Store the ampoules in the original packaging and keep them away from sun light.

| COCAINE<br>(ref. IDT-COP) | HEROIN<br>(ref. IDT-HEP) | MDMA<br>(ref. IDT-ECSP) | MDMA<br>(ref. IDT-ECSP) |
|---------------------------|--------------------------|-------------------------|-------------------------|
| Test + 5 mn               | Test + 5 mn              | Test + 5 mn             | Test + 30 mn            |

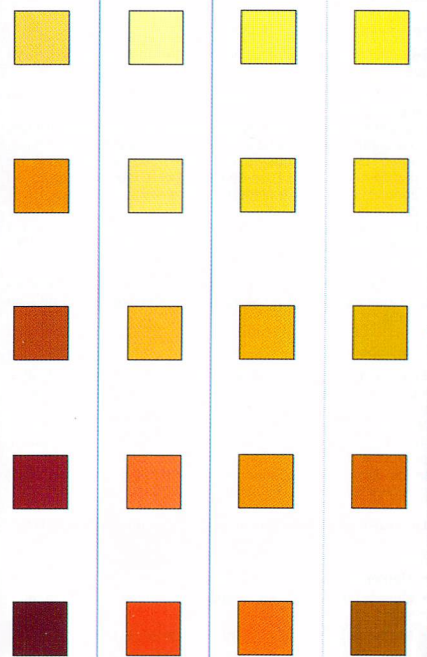Very low purity level / Très faible niveau de pureté  
Sehr niedriger Drogengehalt / Nivel de pureza muy bajo / Livello di purezza molto basso

Low purity level / Faible niveau de pureté  
Niedrige Drogengehalt / Nivel de pureza bajo  
Livello di purezza basso

Medium purity level / Niveau de pureté moyen  
Durchschnittlicher Drogengehalt / Nivel de pureza medio / Livello di purezza medio.

High purity level / Niveau de pureté élevé  
Hoher Drogengehalt / Nivel de pureza alto  
Livello di purezza alto

Very high purity level / Niveau de pureté très élevé  
Sehr hoher Drogengehalt / Nivel de pureza muy alto  
Livello di purezza molto alto

- Keep the ampoules out of reach from children.
- To avoid all risks of cut, use the plastic protective cap to open the ampoule.
- Avoid eye or skin contact with the reagent or granulate. This may lead to irritation (first aid measure: rinse with lots of water). Wearing gloves is recommended.
- Do not ingest or inhale the reagents contained in the ampoule.
- Do not use after the expiration date.

#### More information

For more information on the nature and dangers of substances identified by this test, please go to [www.narcocheck.com](http://www.narcocheck.com) or send us your question by email to [id-test@narcocheck.com](mailto:id-test@narcocheck.com).

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## English

#### Intended use of NarcoCheck® Drug purity ID-Test®

The NarcoCheck® ID-Test® for Drug Purity is a rapid test that gives you an idea about the purity of a drug sample (cocaine, heroin or MDMA). This test helps to find out whether the substance has been abused and just how much of the analyzed drug has been cut. The substances used for cutting may be harmless (sugar, flour, etc.), but it can occur in some circumstances that potentially dangerous substances are added (levamisole, phenacetin, ephedrin, etc.). NarcoCheck® ID-Test® is intended to be used by health professionals, law enforcement, associations that promote prevention for drugs consumption related risks and any person wishing to evaluate a suspect substance.

#### Instructions

1. Take the ampoule out of the plastic tube and prepare a small amount of the substance to identify. You may use the spatula (included in the kit) by covering its hollow interior with powder. If you have a precision scale, measure 20mg. Make sure the powder does not contain any lumps "rocks", as it might distort the results of the test. You can break in powder the compact pieces with the small hammer located at the end of the spatula provided.
2. Hold the ampoule in the upright position and snap off the top plastic part, using index finger and thumb (see picture 1).
3. Only for the Cocaine Purity test: open the buffer solution bottle (included in the kit) and insert the liquid into the test ampoule.
4. By maintaining the opened ampoule in the upright position, insert the sample into the ampoule as indicated on picture 2.
5. Carefully place the plastic cap on the opened ampoule. BEWARE: make sure that the ampoule is well away from your face, because it can happen that the cap pops up under the pressure of the chemical reaction.
6. Gently shake and turn the ampoule to allow the sample and the reagent/granule to mix.
7. Wait 1 minute for the Cocaine Purity test. Wait 5 minutes for the Heroin Purity and MDMA Purity tests.
8. Carefully shake the ampoule for 5 seconds to make the color

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