
NarcoCheck

ID-Test®

Drug Identification Test

www.narcocheck.com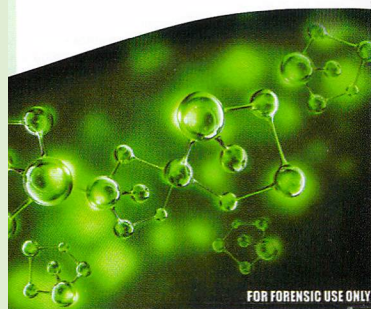

CANNABIS / THC

REF: IDT-THC-xx

	THC, Hashich, Marijuana		Negative
	Instant bright red / Rouge vif immédiat		Négatif
	Rojo vivo inmediato / Sofortiges Hellrot		Negativo
	Rosso vivo immediato		Negativ

COCAINE

REF: IDT-COC-xx

		Cocaine
--	--	---------

COCAINE ADULTERANTS

REF: IDT-CADU-xx

	Ibuprofen		Vitamin-C		Levamisole
	Ephedrine		Lidocaine		Benzocaine
	Phenacetine, Aspirin, Acetaminophen (paracetamol)		Procaine		

DESIGNER DRUGS

REF: IDT-DDR-xx

		Camfetamine, Fluro(meth)amphetamine (4-FA, 4-FMP, 4-FMA) & 2-Aminoindan (2-AI)			MDAI
		Methiopropamine (MPA)			5-APB, 6-APB
		Ethylphenidate			

LSD & TRYPTAMINES

REF: IDT-LSD-xx

		LSD & DMT, DET (2-Me-DET, 4-HO-DET), Ergotamine, 5-MeO-DIPT, 5-MeO-MIPT, 5-MeO-AMT, 5-MeO-DMT, 5-MeO-DALT
--	--	---

ECSTASY / MDMA

REF: IDT-ECS-xx

+ (Meth) Amphetamine

		Ecstasy (MDMA)			Amphetamine MethAmphetamine
--	--	----------------	--	--	-----------------------------

FENTANYL / OXYCODONE

REF: IDT-FENOX-xx

	Fentanyl / Oxycodone
--	----------------------

GHB / GBL

	GHB		GBL
REF: IDT-GHB-xx		REF: IDT-GBL-xx	

HEROIN / MORPHINE

REF: IDT-OPI-xx

	Heroin		Opium		Morphine		Codeine
--	--------	--	-------	--	----------	--	---------

KETAMINE

REF: IDT-KET-xx

	Ketamine & PCP (MeO-PCP, Methoxetamine, N-Ethyl-Ketamine, Tiletamine)		Negative
			Négatif
			Negativo
			Negativ

KHAT (KAT, QAT, GHAT, CHAT)

		Fresh Khat	REF: IDT-FKH-xx
		Dried Khat	REF: IDT-DKH-xx

PMA / PMMA

REF: IDT-PMA-xx

		PMA/ PMMA			Ecstasy (MDMA)
--	--	-----------	--	--	----------------

PRESCRIPTION DRUGS

REF: IDT-PDR-xx

	Oxycodone (OxyContin®, Percocet®)		Methylphenidate (Ritalin®)
	Methaqualone (Mandrax®)		Pethidine (Demerol®)

PSILOCYBIN & PSILOCIN

REF: IDT-PSIL-xx

	Psilocybin		Psilocin
--	------------	--	----------

SYNTHETIC CANNABINOIDS

REF: IDT-SYCA-xx

	JWH-018, JWH-019, JWH-073, JWH-081, JWH-122, AM-2201, AB-001, UR-144, Mix*		AM-2233, JWH-200 RCS-4, XLR-11, Mix*
	JWH-250, AKB-48 & STS-135		URB-597
	AM-694		CP-47, 497
	CP-55, 940		


* mixtures of several cannabinoids / mélanges de plusieurs cannabinoïdes / mezclas de varios cannabinoïdes / Mischungen aus mehreren Cannabinoiden / miscela di diversi cannabinoïdi

SYNTHETIC CATHINONES

REF: IDT-SCAT-xx

	Mephedrone (4-MMC) Pentetone, Butylone, α-PVP Naphyrone, Buphedrone		MDPV
	3-MMC, 4-MEC, 4-MeMABP 3,4-DMMC		Methylone (bk-MDMA)

INSTRUCTIONS

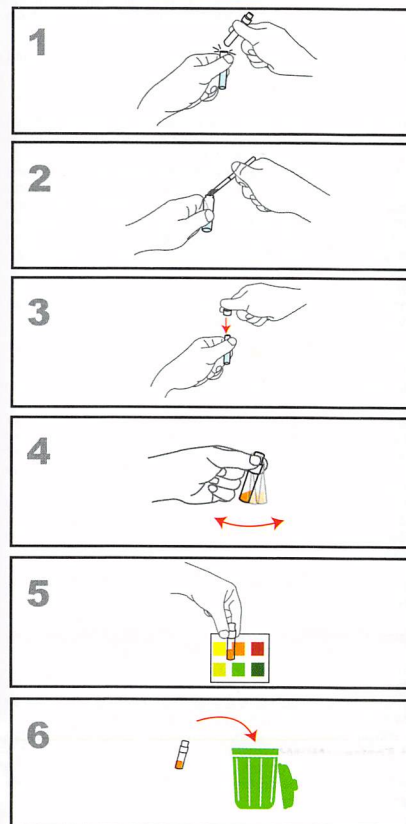English

Intended use of NarcoCheck® ID-Test®

NarcoCheck® ID-Test® is a rapid qualitative test to help identify presumptive drugs or suspicious substances. It is intended to be used by laboratories, law enforcement agencies, associations working to reduce the risks associated with drug use, and anyone looking to identify a suspicious substance.

Instructions

1. Prepare 5mg (roughly the size of half a grain of rice) of the substance to identify. You can break in powder the compact pieces with the small hammer located at the end of the spatula provided. Hold the ampoule in the upright position and snap off the top plastic part, using index finger and thumb as indicated on picture n°1.
2. By maintaining the opened ampoule in the upright position, insert the sample into the ampoule as indicated on picture n°2. **Note** : some ID-Test® (THC, DKH) have a small granule inside of the plastic cap which is a catalyst and also needs to be added into the ampoule.
3. **Carefully** place the plastic cap on the opened ampoule, as indicated on picture n°3. **BEWARE**: make sure that the ampoule is well away from your face, because it can happen that the cap pops up under the pressure of the chemical reaction.
4. **Gently** shake and turn the ampoule to allow the sample and the reagent/granules to mix, as indicated on picture n°4. Any color change should be fully developed within 1 minute.
5. Compare the color change (if any) with the colors on the color chart included in the leaflet in order to identify the substance, as indicated on picture n°5.
6. You can then dispose of everything into a rubbish bin without any harm to the environment.

If the ampoule changes color :

- either, the color matches one of the colors referenced in the colorimetric legend. The test is then positive for the corresponding drug or substance.

• or, the color is uncertain or different from those referenced in the colorimetric legend. The test can then be simply negative for the substance researched or positive for a combination of several substances. Indeed, the positive reaction to one substance may be hidden or modified by the presence of another, which may produce unexpected colorations.

If the ampoule does not change color:

This means the tested sample does not contain any substance indicated on the color chart.

Important warnings

- Regardless of the result obtained, extreme vigilance is required, as:
- The sample may contain harmful substances not identified or not referenced in the colorimetric legend.
 - It is possible that a procedural or handling error, as well as the unexpected interference of certain substances, such as certain cutting products, may cause an erroneous result. The use of this test must be done with the full knowledge that no rapid test can be 100% reliable.
 - The results obtained by this test must be confirmed by a reference analytical method such as GC/MS and HPLC-MS/MS.
 - This test can not give any indication on the quantity or proportion of active ingredient in the sample analyzed.

For these reasons, NarcoCheck® cannot be held responsible for any damage or accident that may result from the use of this test.

The possession and use of drugs is prohibited by law.

Important note:

This test is a professional device to help identify presumptive samples of drugs or narcotics in the form of powders (fine or agglomerated) crystals, tablets, plants or liquids. It is not intended to test «anything and everything» (food products, beverages, biological materials etc.), and in particular everyday products (household, cosmetics, gardening, etc.), which contain chemical compounds that can cause unexpected color reactions.

Storage advice:

- Store these tests in their original packaging, below 35°C (95°F) and protected from sun light.
- Keep out of reach of children.
- To avoid any risk of cutting, use the protective plastic tip to open the ampoule.
- **Avoid eye, mouth, nose or skin contact with the reagents contained in the ampoule, which may cause irritation (first aid measure: rinse thoroughly with water). Use gloves.**
- Do not ingest or inhale the reagents contained in the ampoule.
- Do not use after the expiration date.

More information

For more information on the nature and dangers of substances identified by this test, please go to www.narcocheck.com or send us your question by email to id-test@narcocheck.com.

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